

**Fixation device** used for **securing catheter tubing** onto the patient's body to help prevent injury and maintain comfort.



Sterilized by ETO Gas | Latex free | Do not reuse | Consult instructions for use | Do not re-sterilize | Do not use if package is damaged | Keep away from sunlight | Keep dry

REF: LB-F01 (For Urethral Catheters)  0197



## 1. Intended Use

LockBand®, LB-F01 is non-woven fixation device used for securing catheter tubing onto the patient's body to help prevent injury and maintain comfort. Please read this leaflet before using LockBand®




## 2. Instructions For Use

Your healthcare professional will advise you the best place to position LockBand® to safely secure the catheter and ensure comfort.


### Step 1:

Before applying LockBand® gently clean the skin and pat dry the area where the LockBand® will be applied. Avoid using moisturizing creams or soap to the application area as this will affect the adhesiveness.

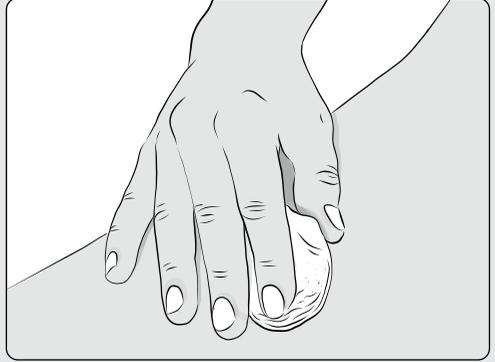


### Step 2:

Remove LockBand® from sterile package and peel away backing protection paper to reveal sticky back.

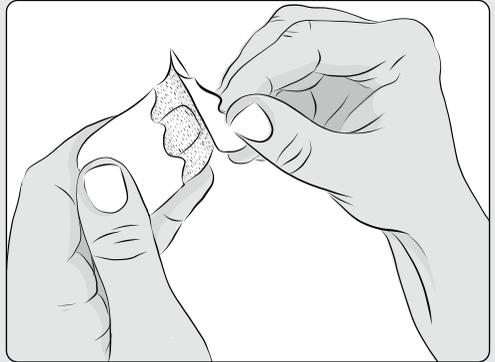


### Step 3:

Apply carefully to the skin.

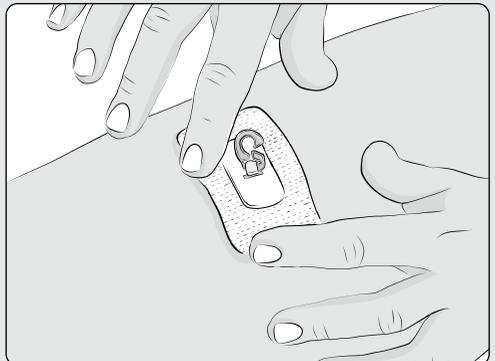

#### Step 4:


Insert catheter balloon inflation tube into catheter holder.

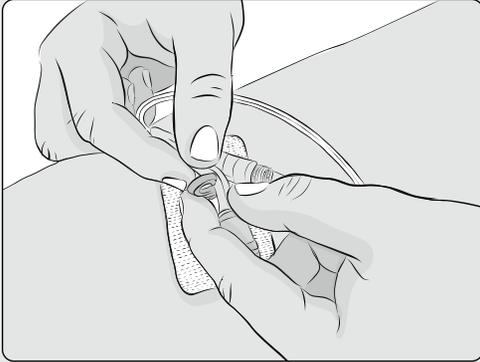

**Recommendation:** It is recommended that you regularly change the site on the leg or abdomen where LockBand® has been placed. This will help maintain good skin integrity and your healthcare professional will advise what is best for you.

#### Step 5:

To remove, release the catheter from the clip and carefully place to one side. Using thumb and forefinger slowly lift and peel the non-woven adhesive backed material starting at one corner. Stop and do not remove if resistance is felt.

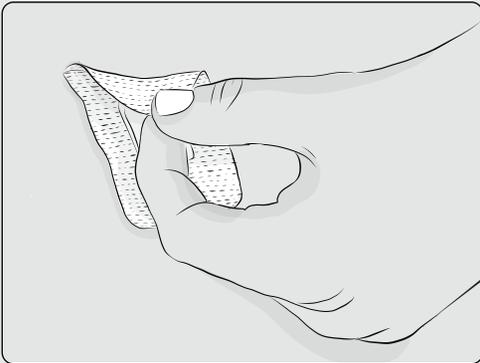

**TIP:** Soak the LockBand® site when bathing/washing and gently peel off to remove LockBand® from the skin.

### 3. Important Product Information

**Store** in original packaging

**Do not** use device if package has been damaged or opened. Check expiry date.

**Dispose the device safely.** Your healthcare professional can advise you.

**Do not** expose the product to high temperature, humid air and ultraviolet ray during storage.

**Do not** re-sterilize this device to avoid any potential risk of infection.

**Do not** use on patients with a known tape or adhesive allergy.

As a general guide the **maximum use is 7 days** or change earlier if product becomes soiled. Your healthcare professional will advise you what is best for you.

### 4. Contraindications

#### Skin Irritation

If skin irritation is observed, fixation tape should be removed from patient's body and seek advice from your healthcare professional. Do not use on unbroken or compromised skin.

#### Caution

Federal (USA) law restricts this device to sale by or on the order of a physician.

### 5. Reorder

To reorder your LockBand® please use the details from the table below:

Product Code	Units Per Pack	Pip Code
LB-F01	10	4060232

### LockBand®



**Yushin Medical Co., Ltd**  
36, Jomaru-ro 427beon-gil,  
Wonmi-gu,  
Bucheon-si, Gyeonggi-Do,  
420-837, Korea

**Tel:** 82 32 668 0323  
**Fax:** 82 32 668 0329



**MPS Medical Products  
Service GmbH**  
Borngasse 20  
35619 Braunfels  
Germany

YMI LB 17 10 LINC 