
The Blood Leak Detection Alarm
for peace of mind during dialysis

User Guide

What is the HEMOalert?




- ▶ The HEMOalert is a blood leakage sensor and alarm for patients undergoing dialysis.
- ▶ It assists in detecting blood leakages from the vascular access. Venous needle dislodgement during haemodialysis is a potentially lethal event due to haemorrhaging.
- ▶ The HEMOalert can be used in a hospital or by a patient at home.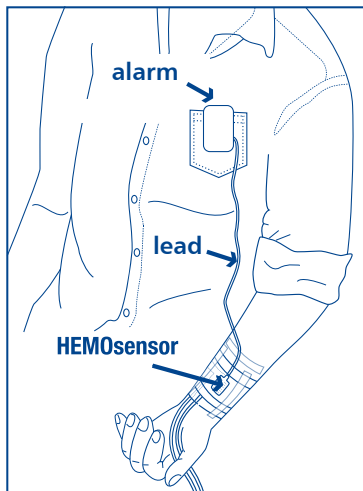

What it consists of

The HEMODialert has three parts:

- ▶ A clip-on alarm that easily attaches to the clothing.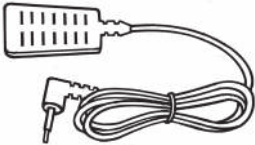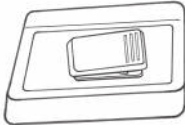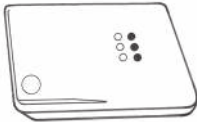

- ▶ A **Reusable** sensor that plugs into the alarm (component name: HEMOsensor).

- ▶ Four, 1.5 volt, Alkaline, button cell batteries that power the alarm.


How it works

If blood leaks from the vascular access and wets the HEMOsensor, an alarm is triggered alerting the user of a possible problem.

How to use the HEMOAlert


1. Always disinfect the HEMOsensor and test the alarm and batteries before use:
 - a) Plug the HEMOsensor into the alarm. If the alarm beeps once, the batteries are functioning.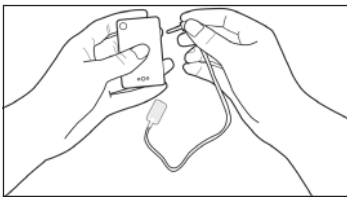

NOTE: If the alarm does not trigger when the HEMOsensor is plugged in then the batteries should be replaced (see section on **How to replace batteries**).

NOTE: When the batteries need replacing the alarm will start to beep slowly at 10 second intervals (approximately) during use.

- b) Use an isopropyl alcohol or hydrogen peroxide swab to disinfect the HEMOsensor and test it is functioning. The alarm will trigger when the HEMOsensor is being swabbed to indicate that it is operational.

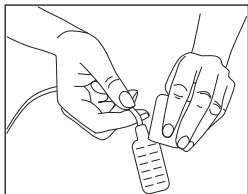NOTE: If the alarm does not trigger when the HEMOsensor is being swabbed, replace the HEMOsensor.

NOTE: Use **only** isopropyl alcohol or hydrogen peroxide to disinfect the HEMOsensor. Do not use chlorhexidine or bleach-based products as these may damage the HEMOsensor.

- c) Switch off the alarm by unplugging the HEMOsensor.
2. Clip the alarm to the patient's clothing e.g. a pocket, a lapel, or the end of a sleeve.


3. Plug the HEMOsensor into the alarm.

NOTE: Ensure the HEMOsensor beeps once when you plug it in.

4. Attach the disinfected HEMOsensor over the vascular access:
- a) If dialysing using an arteriovenous fistula, position the HEMOsensor close to the venous cannulation site.
- b) Tape the sensor in place.

NOTE: Always follow the protocols of the clinic or hospital supervising the dialysis.

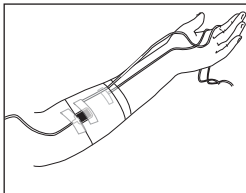5. When the HEMOsensor comes into contact with blood, the alarm triggers until the HEMOsensor is unplugged from the alarm.
6. Always disinfect the HEMOsensor after it comes into contact with blood (see section on **How to clean the HEMOdiAlert**).
7. When not in use, always keep the HEMOsensor unplugged from the alarm to conserve battery life.

How to clean the HEMOdiAlert

Alarm: Wipe clean with a disinfecting swab. **NEVER** immerse the alarm in liquid as this will irreparably damage the alarm.

HEMOsensor: Use an isopropyl alcohol or hydrogen peroxide swab to disinfect the sensor.

NOTE: Use only isopropyl alcohol or hydrogen peroxide to disinfect the sensor. Do not use chlorhexidine or bleach-based products as these will damage the sensor.

Batteries

Four x 1.5V LR44 batteries are included with the HEMOdiAlert alarm. Battery life is expected to be 6 months in normal use, providing the HEMOsensor is unplugged when the alarm is not being used.

Low battery warning indicator: When the HEMOdiAlert is in use and starts to beep slowly at 10 second intervals (approximately) this indicates that the batteries need to be replaced. Replace with alkaline batteries only. Always dispose of batteries in an environmentally responsible and safe way.

How to replace the batteries:

- a) Remove the back of the alarm by separating the two raised areas on the side of the alarm.
This is easy to do by inserting any thin rigid item between them and twisting it gently.
- b) Take care not to touch the internal circuitry.
- c) Insert four new batteries as indicated by the diagram inside the alarm.

Troubleshooting

Problem: The alarm does not beep when the HEMOsensor is plugged in.

Solution 1: Replace the batteries. If this does not solve the problem proceed to solution 2.

Solution 2: Replace the HEMOsensor. If this does not solve the problem proceed to solution 3.

Solution 3: Replace the alarm unit.

Problem: The alarm beeps when the HEMOsensor is plugged in but it does not trigger when the HEMOsensor is subsequently disinfected.

Solution: Replace the HEMOsensor. The HEMOsensor is a consumable item and we recommend that it is replaced every 6-9 months.

Problem: The alarm triggers when it detects sweat.

Solution: Wrap the HEMOsensor in sterile gauze before attaching it over the vascular site.

NOTE: Always follow the protocols of the clinic or hospital supervising the dialysis.

Manufacturer

Made in New Zealand by Anzacare Limited,
P O Box 400, 13 Mahara Place, Waikanae 5036
Email: info@anzacare.com

Warranty

The manufacturer will replace any alarm which fails due to manufacturing fault within 6 months of purchase. This warranty does not cover batteries or misuse or accidental damage to the alarm. Contact info@anzacare.com if you have a warranty claim.

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Hergestellt in Neuseeland von Anzacare Limited,