



### **FIXSTRIP - Directions for use**

FIXSTRIP is designed to be worn around the leg or abdomen to secure the catheter preventing pulling of the catheter

Place FIXSTRIP around the leg or abdomen gently stretching to give a comfortable fit. Position so the catheter fixation area is to the front of the abdomen of thigh. Ensure the two silicone grippers are facing down onto the skin and secure with the Velcro fastener. Use the scissors provided to cut off any excess strapping.

Secure the catheter is the fixation straps as per the diagrams below:



The FIXSTRIP should be changed every 5-7 days or whenever soiled or damaged.

The FIXSTRIP should be checked regularly to ensure the catheter strap is not too tight.

We recommend the FIXSTRIP is removed at bedtime and follow your usual overnight drainage regime as recommended by your Healthcare Professional.

The FIXSTRIP can be used in the shower, is machine washable up to 60 degrees and can be tumble dried.

Ensure you have sufficient supplies of FIXSTRIP. Further supplies can be obtained through your Healthcare Professional or alternatively contact our Customer Service Team on 01572 717515 quoting product code FIXSTRIP.

Catheters marketed in the UK by L.IN.C Medical Systems Ltd. are manufactured by Shanghai Jianying Medical Technology Co., Ltd Area B, 4th Floor, Building 11, Lane258, Yinlong road, Jiading district, Shanghai,201806 China.

Leaflet reference number 09/2016-v1



7 Kingsley Street, Leicester, LE2 6DY  
Telephone: +44 (0) 1572 717515 Facsimile: +44 (0) 1572 717516  
E-mail: [sales@linc-medical.co.uk](mailto:sales@linc-medical.co.uk) Website: [www.linc-medical.co.uk](http://www.linc-medical.co.uk)