



**BIK allows clinicians to easily administer a catheter maintenance solution without the need to disconnect the catheter and drainage bag and break the sterile system.**

### Instructions for Use

#### The 5 simple steps to BIK:



Wash hands



Clamp the bag tubing below the sample port with the red clamp



Clean the sample port according to local policy and insert blue funnel into sample port.



Insert catheter maintenance solution pack into funnel securely and follow manufacturer instructions.



When instillation is complete, remove red clamp and allow solution to drain into bag. Finally, remove BIK funnel and dispose according to