

LINC Night Bags



LINC-Flo Reusable

LINC-Flo reusable night drainage bags allow problem free drainage to help patients have a good nights' rest.



Long flexible tubing and needle free sample port

Integrated bed hanger

TZ02

Approx vol. ml.
2000
1800
1600
1400
1200
1000
800
600
400
300
200
100

100
75
50
25

Each bag includes:
SLC13

Nitrile gloves

T-tap and lever tap available

Single use Night Bags



Linc Night bags come with either a single use or reusable taps.

Instructions for Use:

Connection

- Wash hands. Take out the bag from its packaging and check that the taps are in the closed position.
- Remove connector cap and insert the ridged connector fully for a secure connection into the catheter funnel for reusable taps or leg bag funnel for single use taps.
- Ensure that the night bag is positioned below the leg bag or bladder and supported by an appropriate hanger or stand
- Ensure that any of the tubing is NOT kinked and the leg bag tap is open (if connected to one).
- All LINC Night Bags are for single use only and must not be reused once disconnected.

Emptying

- Empty the night bag by opening the tap over a toilet or designated receptacle.
- All LINC bags are ready to use and come with the taps closed
- **Lever tap:** open the tap by pushing the lever downwards towards the outlet tap and close by pulling the lever tap upwards towards the bag
 - o The lever points towards the desired flow of urine
 - o Tap starts to open/close after the 90° angle of travel
 - o The whole tap section may be twisted to avoid accidental opening
- **T-tap:** open and close the tap by pushing the barrel sideways
- Remember to close the tap after emptying.
- Ensure that the outlet tap is as dry and without any residual urine as possible
- **Twist tap:** turn the bag upside down with the tap pointing up; using both hands, hold the tap and twist-off the outermost tip; invert the bag to allow the urine to flow out into the toilet/receptacle

Usage and Disposal

- LINC night bags with re-usable taps should be changed between 5 to 7 days of use.
- LINC night bags with single use taps should be disposed of after emptying.
- Store at room temperature
- Dispose with normal household waste or as directed by your clinician
- Do not flush down the toilet

Tap type	Use	Sterile	Capacity
Twist Tap	single use	non sterile	2L
	single use	sterile	2L
	single use	sterile	3L
T-Tap	single use	sterile	2L
	re-usable	sterile	2l
	re-usable	sterile	3L
Lever Tap	re-usable	sterile	2L

Distributed in the UK by LINC Medical. **Manufactured** by Changshu Wilson Medical Devices Co., Ltd., No 1 Taikang road, North of Zhitang Town, Changshu City, 215531 Jiangsu Province, China. **EC Representative:** Welkang Ltd, Suite B, 29 Harley Street, London W1G 9QR, UK.